

RELEASE417 - INSTRUCTIONS FOR USE

4 steps to Release



- 1: Place brace on your thigh with the logo facing up | 2: Pull Velcro strip through ring
- 3: Pull firmly to tighten | 4: Securely fasten brace using Velcro strip

Determine your size:

	S	M	L	Extender
	42 - 49 cm	49,5 - 56,5 cm	57 - 64 cm	max + 5 cm

Met opmerkingen [1]: Hieronder in de tabel moeten de comma's met punten vervangen worden (b.v., 49,5 > 49.5 etc.)

If the circumference of your thigh, measuring one hand-width from your groin, is close to the upper limit of a particular size range, you can always use the extender. This will enable you to extend the brace by a couple centimetres. This will also make it easier to pull the strip through the ring and tighten it. Only use the extender if this is really necessary.

Washing instructions:



Press the ends of the Velcro strip together and wash the product by hand in lukewarm water. You can use a mild shampoo as a detergent. Do not dry in clothes dryer | Do not dry-clean | Do not iron | Do not use bleach

WARNINGS:

- Do not use the brace on your arms. It is not suitable for this purpose.
- Do not use the brace on your neck or other body parts.
- If you experience discomfort immediately after putting the brace on, loosen it a bit around your leg or remove it entirely.
- Be careful using the brace if you suffer from vascular disorders.
- Be careful using the brace if you suffer from thrombosis. Consult your doctor.
- If you wear the brace while walking, make sure that it is tight enough so that it doesn't slip off to prevent tripping or falling.
- Make sure the Velcro strip is properly closed and doesn't stick out too far since the hooks on the Velcro strip can get caught in clothing or bed linens.
- The product must be used according to the instructions above, and taking the warnings into account.
- Release417 is not liable for damage resulting from the improper use of the product.
- If you suspect there are manufacturing defects in the product after purchasing it, do not use the product and contact the supplier.

Materials:

- Elastane | Polyester | Plastic (POM) | Polyester Velcro | PU foam | Silicone

www.Release417.com | info@release417.com | Release417 is a registered trademark of Release417